VERMEIREN

Lyna II - Lindsey

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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EN**Contents**

1. Technical details	2
2. Components	2
3. Explanation of symbols	3
4. Check after delivery	3
5. Intended use	3
6. Before use	3
7. Fold and unfold the walking frame (Fig. 1)	3
8. Height adjustment handgrips (Fig. 2)	4
9. Carrying walking frame	4
10. Changing to articulating walking frame (Fig. 3 and 4)	4
11. Walking with non-articulating walking frame	4
12. Walking with articulating walking frame (only for Lyna II walking frame)	5
13. Transfer wheelchair to walking frame Lyna II	5
14. Transfer wheelchair to walking frame Lindsey	5
15. Safety instructions	5
16. Maintenance, inspection, cleaning	5

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Walking frame Lyna II, Lindsey

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Walking frame 
Model	Lyna II
Total width	540 mm
Total length	470 mm
Maximum height	960 mm
Minimum height	780 mm
Width (folded)	120 mm
Length (folded)	510 mm
Height (folded)	780 mm - 955 mm
Distance between handgrips	495 mm
Weight	2,65 kg
Max. inclination forwards	10°
Max. inclination backwards	5°
Max. inclination sideways	8°
Maximum user weight	max. 130 kg
Turning diameter	1180 mm
Maximum angle handgrips	0° (not articulating) 3,5° (articulating)
Storage temperature	+5°C - +41°C
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / °	

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Walking frame 
Model	Lindsey
Total width	590 mm
Total length	540 mm
Maximum height	970 mm (handgrip 1) 710 mm (handgrip 2)
Minimum height	790 mm (handgrip 1) 535 mm (handgrip 2)
Width (folded)	120 mm
Length (folded)	605 mm
Height (folded)	790 mm - 970 mm
Distance between handgrips	475 mm
Weight	2,75 kg
Max. inclination forwards	10°
Max. inclination backwards	5°
Max. inclination sideways	8°
Maximum user weight	max. 130 kg
Turning diameter	1510 mm
Maximum angle handgrips	0° (not articulating)
Storage temperature	+5°C - +41°C
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / °	

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.



Before using the walking frame, read the instruction manual carefully: it will familiarise you with this product.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components

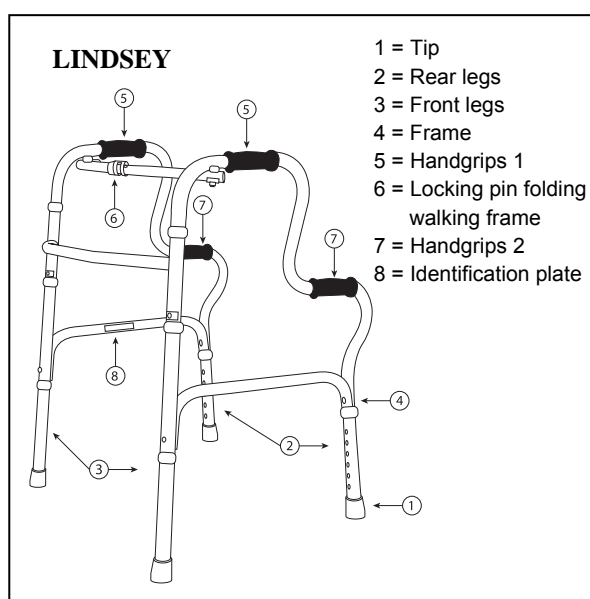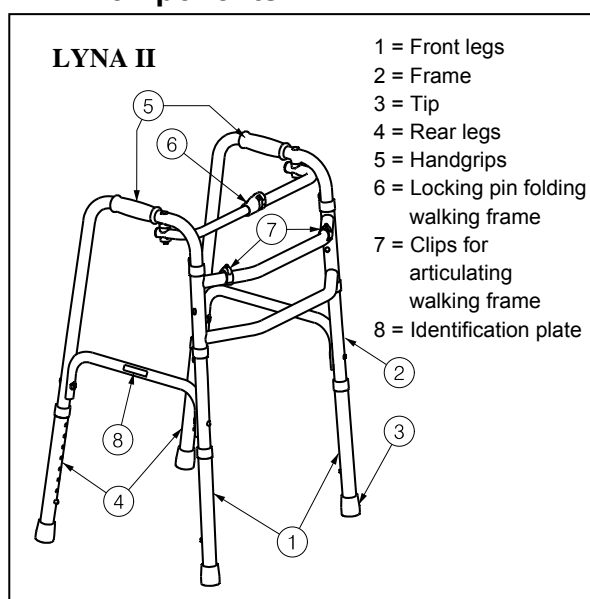3. Explanation of symbols

Maximum mass

CE conformity

Type designation

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- walking frame with 4 tips (mounted)
- height adjustable handgrips
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

⚠ WARNING: Risk of falling - Do not use the walking frame if you have weak arm function or poor equilibrium.

The walking frame will compensate the lack of adequate support by the legs by leaning on, or moving with, the walking frame with usage of the upper limbs or chest. The walking frame will provide a better stability, support and safety during walking.

However, it cannot be used by persons with a visible weakened hand and/or arm function and also not by persons with a balance problem.

The walking frame can be used indoor and outdoor.

The walking frame should only be used on surfaces where all four tips are touching the ground. Not to be used on rocky terrain or steep hills (see technical details: max. inclination) so the walking frame can become unstable.

The product is not meant for transport of people and/or goods.


6. Before use

- Remove all packaging before using the walking frame for the first time.
- Unfold the walking frame.
- Adjust the handgrips in the desired height.

7. Fold and unfold the walking frame (Fig. 1)

⚠ WARNING: Risk of injury - Do not press on the locking pin (① in Fig. 1) unless you fold the walking frame.

⚠ WARNING: Risk of injury - Check that the walking frame is completely unfold before using it.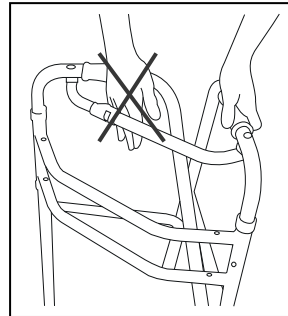

⚠ WARNING: Risk of clamping - Keep fingers, buckles and clothes away from the entrapment zone.

To transport the walking frame in the car or airplane you can fold the walking frame.


1. Folding walking frame

- Press the locking pin ① in the middle of the bar ②.
- Grasp the handgrips.
- Fold the side frames ③ one by one, inwards.

2. Unfolding walking frame

- Grasp the handgrips.
- Fold the side frames ③ one by one, outwards until it clicks in position.
- Check that the side frames ③ are well fixated and cannot move inwards, before using the walking frame.

Fig. 1 – Folding / Unfolding walking frame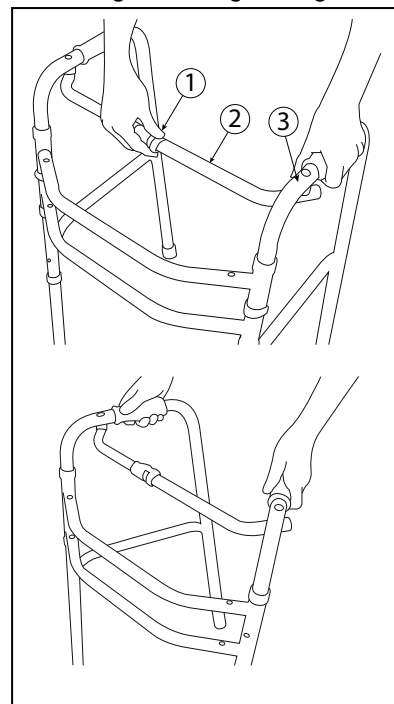


8. Height adjustment handgrips (Fig. 2)

- ⚠ **CAUTION: Risk of falling - Secure the handgrips firmly before using the walking frame. Check that the spring pin ① is firmly fixated in a hole.**
- ⚠ **CAUTION: Risk of falling - Do not exceed the maximum height adjustment of the handgrips, your walking frame will not be stable anymore.**
- ⚠ **CAUTION: Risk of tipping over - Place the four legs at the same height.**

Before using the walking frame you must adjust the handgrips in the correct height.

Fig. 2 – Adjustment handgrips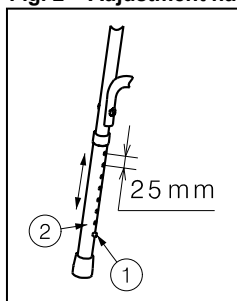

The handgrips can be adjusted in height by following steps:

- Press on the spring pin ①.
- Move the leg ② until the desired height (8 different positions: steps from 25 mm).
- Check that the spring pin ① is clicked back and firmly fixated in a hole.
- Adjust the other legs with the same instructions.
- Check that all legs are placed in the same position. When standing straight, the handgrips should be as high as your wrists. Leave your arms hanging loose. Elbows should be flexed at 20° to 30°.

9. Carrying walking frame

The best way to carry the walking frame is to grasp the walking frame on the handgrips when the walking frame is folded or unfolded.


10. Changing to articulating walking frame (Fig. 3 and 4)

For the Lyna II walking frame there is the possibility to change these to an articulating or non-articulating walking frame.

Adjust the walking frame to articulating walking frame (Fig. 3):


- Remove the locking clips ① out the holes of the side tubes ② on the front of the walking frame.
- Place the locking clips ① in the holes of the top tube ③.
- Now the side frames of the walking frame can be articulated (0° - 3,5°). The advantage is that these give you always a stable support.

Fig. 3 – Articulating walking frame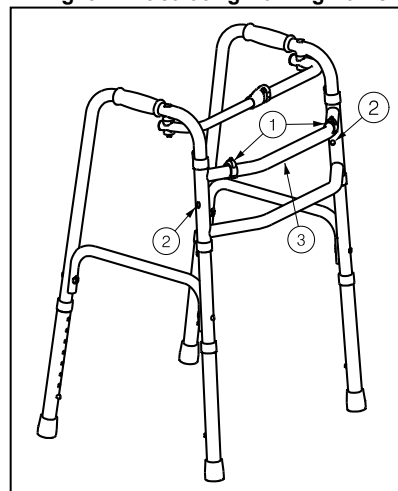

Adjust the walking frame to non-articulating walking frame (Fig.4):


Fig. 4 – Non-Articulating walking frame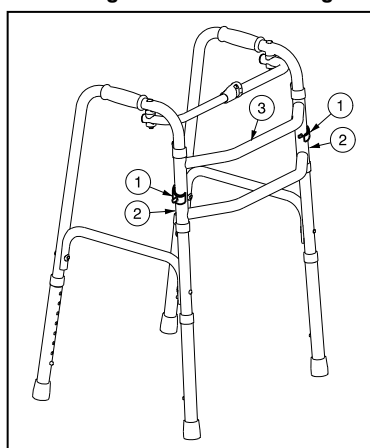

- Remove the locking clips ① out the holes of the top tube ③.
- Replace the locking clips ① in the holes of the side tubes ② on the front of the walking frame.

- Now the side frames of the walking frame cannot articulated (0°).

11. Walking with non-articulating walking frame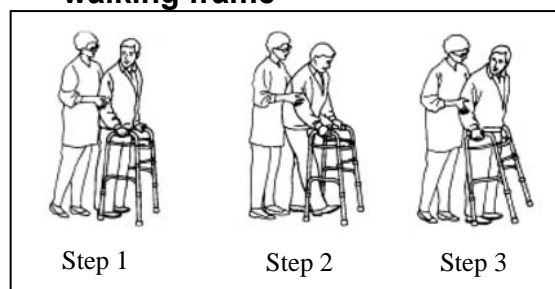

Step 1:
Align the middle of your feet with the rear legs of the walking frame.

Step 2:
Lift the walking frame up and place it in front of you making sure all four legs end up on the floor.

Step 3:
Carefully walk forward one step at a time.

12. Walking with articulating walking frame (only for Lyna II walking frame)

Step 1:


Align the middle of your feet with the rear legs of the walking frame.

Step 2:


Slide one side frame with the handgrip forward. Afterwards slide the other side frame with the handgrip forward.

13. Transfer wheelchair to walking frame Lyna II

Make certain the wheels of the wheelchair are locked. Lift your body by pushing down on the armrests and carefully transfer, one hand at a time, to the walking frame (ask assistance if you feel uncertain).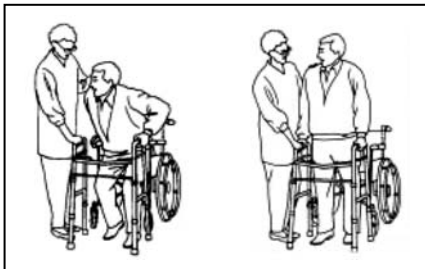

(with lower extremity injury to right leg or foot) Make certain the wheels of the wheelchair are locked. Lift your body by pushing down on the armrests and carefully transfer the left hand first, followed by the right hand (If the injury is on the left leg or foot, transfer the right hand first, followed by the left).

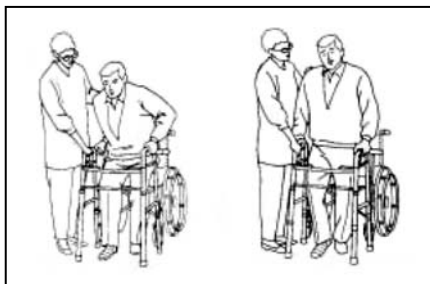14. Transfer wheelchair to walking frame Lindsey

The walking frame Lindsey have less high handgrips to get easier and safer out the wheelchair, bed,

Make certain the wheels of the wheelchair, bed, ... are locked. Lift your body by pushing down on the armrests and carefully transfer, one hand at a time, grasp first the less high handgrips of the walking frame, afterwards the higher handgrips.

15. Safety instructions

- Check that the walking frame is in a perfect condition before using it. If there

are damaged/missing parts change these immediately by a qualified technician.

- Do not sit on your walking frame - Risk of tipping over or injury.
- Use only the handgrips as support.
- Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- Before every usage, check that the locking pins for height adjustment of your handgrips are firmly locked.
- Verify if the safety against unwanted folding/unfolding is functioning properly.
- Only proceed moving with your walking frame when you are standing comfortably and when both your hands are resting on the handgrips.
- Do not hold the handgrips with wet hands. If you do so, you can loose your grip and loose your balance.
- When you detect that your body is getting tired, check that you can find a place for resting.
- **The maximum user weight for your walking frame is 130 kg.**
- When using the walking frame outdoors, find yourself flat and stable areas to use your walking frame. Avoid curbs and/or deep holes. Find a place where the curbs are beveled.
- Avoid bumpy or sandy roads. Otherwise you might loose control over the stability of your walking frame.
- Be aware of: throw rugs, toys, drapes, animals, cracks in the floor, liquids or puddles.
- Do not proceed with your walking frame into traffic. Always stay on the pavement.
- Avoid roads, slopes with a very steep inclination (the maximum angles are mentioned in the technical details).
- Do not use your walking frame on staircases, escalators.
- Exercise when walking on slope surfaces, wet or slippery surfaces.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

16. Maintenance, inspection, cleaning

For the maintenance manual of the walking frames refer to the Vermeiren website: www.vermeiren.com.

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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All rights reserved, including translation.
Vermeiren Suisse S.A.