User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

Normandie XXL Languedoc XXL

EN

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Verze: A, 2023-01

All rights reserved, including translation.

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Bedienungshandbuch

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Manual de instrucciones

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This page is intentionally left blank

Contents

Contents	1
Preface	2
1 This product	3
2 Before use	4
2.1 Intended use	4
2.2 General safety instructions	4
2.3 Symbols on the chair	4
2.4 Assembly/Disassembly	5
2.5 Transport and storage	5
3 Using this chair	6
3.1 Transfer into and out of the chair	6
3.2 Comfort adjustments	6
3.3 Running wheels / Parking brakes (only with wheels)	7
4 Maintenance	9
4.1 Points of maintenance	9
4.2 Instructions for maintenance	9
4.3 End of use	9
5 Technical specifications	10

Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this comfort chair and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your chair.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without being imposed to any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

This instruction manual
For user and specialist dealer

Service manual
For specialist dealer

EC declaration of conformity

1 This product

1. Base frame
2. Adjustable backrest
3. Removable seat
4. Height adjustable armrests
5. Arm pads
6. Leg rest
7. Inclination lever
8. Transport wheels
9. Identification plate

2 Before use

2.1 Intended use

i In this paragraph a brief description of the intended use of your comfort chair is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.

- The comfort chair is intended to be used by the elderly, during revalidation, or by patients suffering from paralysis.
- This comfort chair is suitable for indoor use only, on flat, level grounds.
- This chair is designed and produced solely as seat and resting place for one (1) person with a maximum weight of 350kg. It is not designed for transport of goods or objects, nor for any other use as previously described.
- If applicable, only use accessories and spare parts approved by Vermeiren.
- Please read all technical details and limits of your chair in §5.
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

2.2 General safety instructions

 CAUTION Risk of injuries and/or damage

Keep the following general warnings in mind during use:

- Be aware that some parts of this chair may get very hot or cold due to ambient temperature, solar radiation, heating devices. Be careful when touching. Use protective clothing if the weather is cold.
- Be careful when using possible causes of fire such as cigarettes since they may set the seat and back covers alight.
- Do not modify this shell chair in any way.

2.3 Symbols on the chair

The symbols in following list are applicable for your chair. Symbols can be found in the relevant ISO standard (ISO 7000, ISO 7001 and IEC 417).

	Maximum weight of user in kg
	Indoor use only
	Type designation

The comfort chair complies to the requirements set up in:

EN 1021-1, EN1021-2: Resistance to ignition of upholstered parts

2.4 Assembly/Disassembly

Upon delivery of the comfort chair, the seat has to be installed before use. The seat can also be removed for easy transport.

2.4.1 Seat

1. Place the hooks on the rear side of the seat over the frame tube close to the backrest.
2. Push down on the front side of the seat to fix it into place. You will hear a click.

To remove the seat, pull of the front of the seat until it detaches from the frame.

2.5 Transport and storage

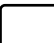
 CAUTION Risk of injuries and damage

- Pay attention when passing with the chair through narrow passages.
- Make sure the passage is clear from objects, obstacles and persons.

2.5.1 Moving aside

1. Check that the brakes of the comfort chair are engaged (only with wheels, see §3.3).
2. Transfer the patient out of the chair.
3. Disengage the brakes (only with wheels).
4. Place the backrest into the upright position.
5. Transport the chair to the desired place on a horizontal underground.
 - a) *With wheels*: Move the comfort chair by using its wheels and roll it to its destination.
 - b) *Without wheels*: Ask help to lift the chair with two persons. Firmly grasp the frame with both hands and lift it from the floor. Do not use leg rest(s), armrests, wheels or seat to grasp the chair.
6. Engage the brakes (only with wheels).

2.5.2 Storage

 CAUTION Risk of damage

Make sure that your comfort chair is stored dry to prevent mould from growing and the upholstery from being damaged, see also technical details in §5.

3 Using this chair

WARNING

Risk of injuries

- First read previous chapters and inform yourself about the intended use. Do NOT use your chair unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you on this.

CAUTION

Risk of injuries and/or damage

- Make sure that your fingers, clothes, belts, buckles or jewellery won't get caught by moving parts, during assembly, use or adjustment.

3.1 Transfer into and out of the chair

CAUTION

Risk of injuries or damage

- In case you cannot perform the transfer in a safe manner, ask someone to assist you.
1. Check that the brakes of the comfort chair are engaged (only with wheels, see §3.3).
 2. Place the comfort chair in its most upright sitting position.
 3. Transfer to/out of the chair. Place the arm pad at the same height as the seat cushion for a sideways transfer (see paragraph 3.2.2).

3.1.1 Correct position in the chair

Sit down on the seat with the lower back against the backrest.

3.2 Comfort adjustments

CAUTION

Risk of injuries or damage

- Apply the brakes before adjustments (only with wheels, see §3.3).

3.2.1 Backrest inclination Normandie XXL/Languedoc XXL

CAUTION

Risk of injuries or damage

- Be sure that the comfort chair does not tip over backwards when it is placed in its most backwards position.
- Make sure that no objects or individuals are in the adjusting area (see entrapment zones).

The angle of the backrest and the leg rest can be adjusted by means of a gas spring.

Entrapment zones are indicated in the figure below by (1).

To incline the comfort chair:

1. Engage the brakes (only with wheels, see §3.3).
2. To incline the backrest and leg rest simultaneously, pull the lever (2) and push against the backrest until you reach the desired position (backrest inclination: 13° - 40°; leg rest inclination: 0° - 80°). Let go of the lever to block the backrest and leg rest into position.
3. To reduce the backrest and leg rest inclination, pull the lever (2) and let the backrest and leg rest return to the desired position. Let go of the lever to block the backrest and leg rest into position.

Minimum inclination

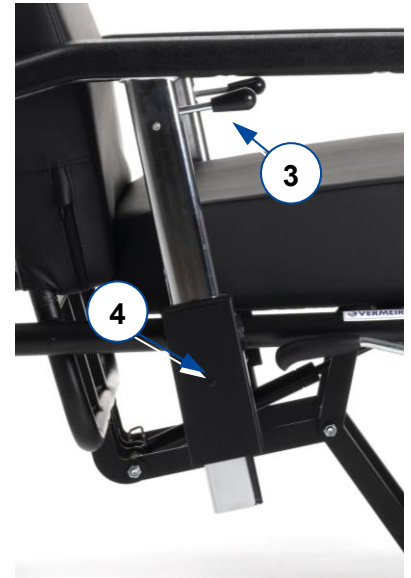
Maximum inclination

3.2.2 Adjusting the armrests

The armrests can be adjusted in height and can be lowered to the height of the seat frame for the sideways transfer of the patient.

1. Pull the lever of the armrests (3) upwards (in the direction of the arm pad).
2. The locking spring will be released and the armrest can be moved within the armrest socket (4) into the desired position (range 114 mm, steps of 19 mm).

CAUTION

Risk of injuries

- If the locking spring is guided outside the armrest socket (4), the armrest is not locked. Use this position only for the sideways transfer of the patient.

For sideways transfer of the patient:

Pull the lever (3) and guide the armrest down to the lowest possible position.

Now the arm pad comes at the same height of the seat cushion.

3.3 Running wheels / Parking brakes (only with wheels)

CAUTION

Risk of injuries or damage

- Lock all four wheels in place to put the comfort chair in a parking position.
- Make sure that the comfort chair is on a flat horizontal surface before releasing the brakes. Never release all the brakes simultaneously.

5

Engaging the parking brakes

With your foot, press the brake plate of the wheels downwards until it locks into place. The wheels cannot roll.

6

Disengaging the parking brakes

With your foot, press the brake plate of the wheels in the direction of the frame. The wheels have now been released and can both roll and swivel.

4 Maintenance

- i** Regular care ensures that your comfort chair is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: www.vermeiren.com.

4.1 Points of maintenance

 CAUTION Risk of injuries and damage

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

- i** The last page of this manual contains a registration form for the specialist dealer to record each service.

The service frequency depends on the frequency and intensity of use. Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

Before each use

Inspect the following points:

- All parts: Present and undamaged or unworn.
- All parts: Clean, see § 4.2.1.
- Wheels, backrest, seat: Well secured.
- Working of gas springs, brakes, armrests.
- Wheels/tyres; keep them free of wires, hair, sand, fibres and other contamination (oil, mud).
- Condition of frame parts: No deformation, instability, weakness or loose connections
- Upholstery, seat cushion, calf-rest cushion: No excessive wear (like dented spots, damage or tears).

Contact your specialist dealer for possible repairs or part replacements.

4.2 Instructions for maintenance

4.2.1 Cleaning

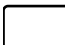
 CAUTION Risk of damage

Never use a steam cleaner to clean the chair.

Wipe all rigid parts of the chair frame with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The upholstery can be cleaned with lukewarm water and a mild soap. Do not use abrasive cleaning agents, sponge, or hard brush to clean.

4.2.2 Disinfection

 CAUTION Risk of damage

Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

4.3 End of use

At end of life, you need to dispose your chair according to the local environmental legislation. The best way to do so, is to disassemble the chair to facilitate the transport of recyclable parts.

5 Technical specifications

The technical details below are only valid for this chair, at standard settings and optimal ambient conditions. Take these details into account during use. The values are no longer applicable if your chair has been modified, damaged, or is severely worn.

Make	Vermeiren
Type	Comfort chair
Model	Normandie XXL
Maximum occupant mass	350 kg

<i>Description</i>	<i>Dimensions</i>	
	<i>Without wheels</i>	<i>With wheels</i>
Overall length	820 mm	
Overall width	890 mm	
Total mass	Approx. 37,75 kg	Approx. 39,3 kg
Mass of heaviest part	33,35 kg	34,9 kg
Masses of parts that can be removed or dismantled	Seat: 4,4 kg	
Seat plane angle	7°	
Effective seat depth	490 mm	
Effective seat width	730 mm	
Seat surface height at front edge	510 mm	520 mm
Backrest angle	13° - 40°	
Backrest height	780 mm	
Armrests height	0 mm (only for sideways transfer) 138 mm - 220 mm	
Overall height	1205 mm	1210 mm
Dismantled height	810 mm	
Thickness seat cushions	100 mm	
Angle between seat and leg rest	0° - 80°	
Storage and use temperature	+ 5 °C - + 40°C	
Storage and use humidity	30% - 70%	
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / 1,5°		

Table 1: Technical specifications Normandie XXL

Make	Vermeiren
Type	Comfort chair
Model	Languedoc XXL
Maximum occupant mass	350 kg

<i>Description</i>	<i>Dimensions</i>	
	<i>Without wheels</i>	<i>With wheels</i>
Overall length	820 mm	
Overall width	890 mm	
Total mass	Approx. 28,7 kg	Approx. 31,35 kg
Mass of heaviest part	24,3 kg	26,95 kg
Masses of parts that can be removed or dismantled	Seat: 4,4 kg	
Seat plane angle	7°	
Effective seat depth	490 mm	
Effective seat width	730 mm	
Seat surface height at front edge	510 mm	520 mm
Backrest angle	13° - 40°	
Backrest height	780 mm	
Armrests height	0 mm (only for sideways transfer) 138 mm - 220 mm	
Overall height	1205 mm	1205 mm
Dismantled height	810 mm	
Thickness seat cushions	100 mm	
Storage and use temperature	+ 5 °C - + 40°C	
Storage and use humidity	30% - 70%	
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / 1,5°		

Table 2: Technical specifications Languedoc XXL

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**Service registration form**

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Vermeiren GROUP
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com