


Use WEICONLOCK AN 302-50

Material			
No.	Plan	Article	Quantity
1	YP08302(R)	2613026KTL	1
	YP08306(L)	2613226KTL	1
2	YP06331(R)	2798326KTL	1
	YP06931(L)	2768926KTL	1
3	YP08304	2613126KTL	1
4	XP01732(R)	2993523	1
	XP01731(L)	2993522	1
5	XP01989	GP13900	1
6	X004747	1906057	2
7	XP02093	GP15800	1
8	XP02047	2993677	1
9	M6x16	2994067	2
10	XP01988	GP13800	1
11	M6x35	2994634	2
12	M6x30	1915911	1
13	XP02815	GP21100	2
14	M6x35	1006057	1
15	M6	2994063	1
16	Washer D6	1001564	1
17	Washer D8	1996214	2
18	M6x40	2994317	1
19	XP03133	GP25000	3
20	P00070611 (R)	2680465	1
	P00071611 (L)	2680565	1

EP00297 / K

ANTI TIPPING FOR TRIGO				Page 1/3	Plan No.
					EP00297 / K
Scale: 1:4	Drafted	Verified	Confirmed		
Date	26/05/2018	06/02/2024	06/02/2024		
Initials	AS	PM	AS		

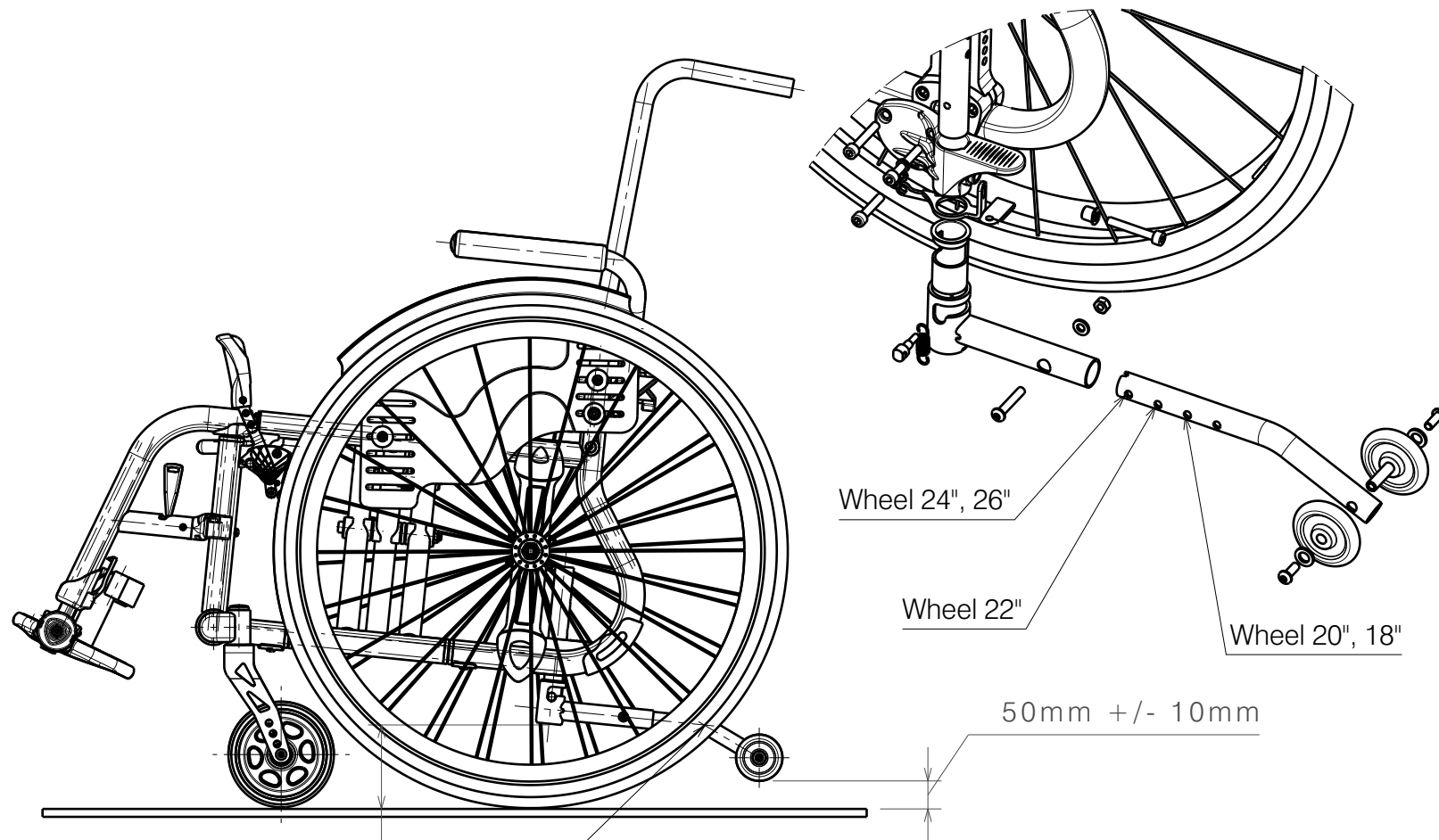


This X dimension should be as large as possible.
 Default position of tube should be bent towards the ground.


/
 Wymiar X powinien być jak największy.
 Domyślne ustawienie rury zagięciem w kierunku podłoża.

Optimal distance between the ground and the small
 antitipper wheel must be 50mm +/- 10mm.

/
 Optymalny wymiar pomiędzy podłożem a kołem
 przeciwwyrotowym wynosi 50mm +/- 10mm.

EP00297 / K


ANTI TIPPING FOR TRIGO				Page 2/3	Plan No.
					EP00297 / K
Scale:	1:4	Drafted	Verified	Confirmed	
Date	26/05/2018	06/02/2024	06/02/2024	06/02/2024	
Initials	AS	PM	AS		

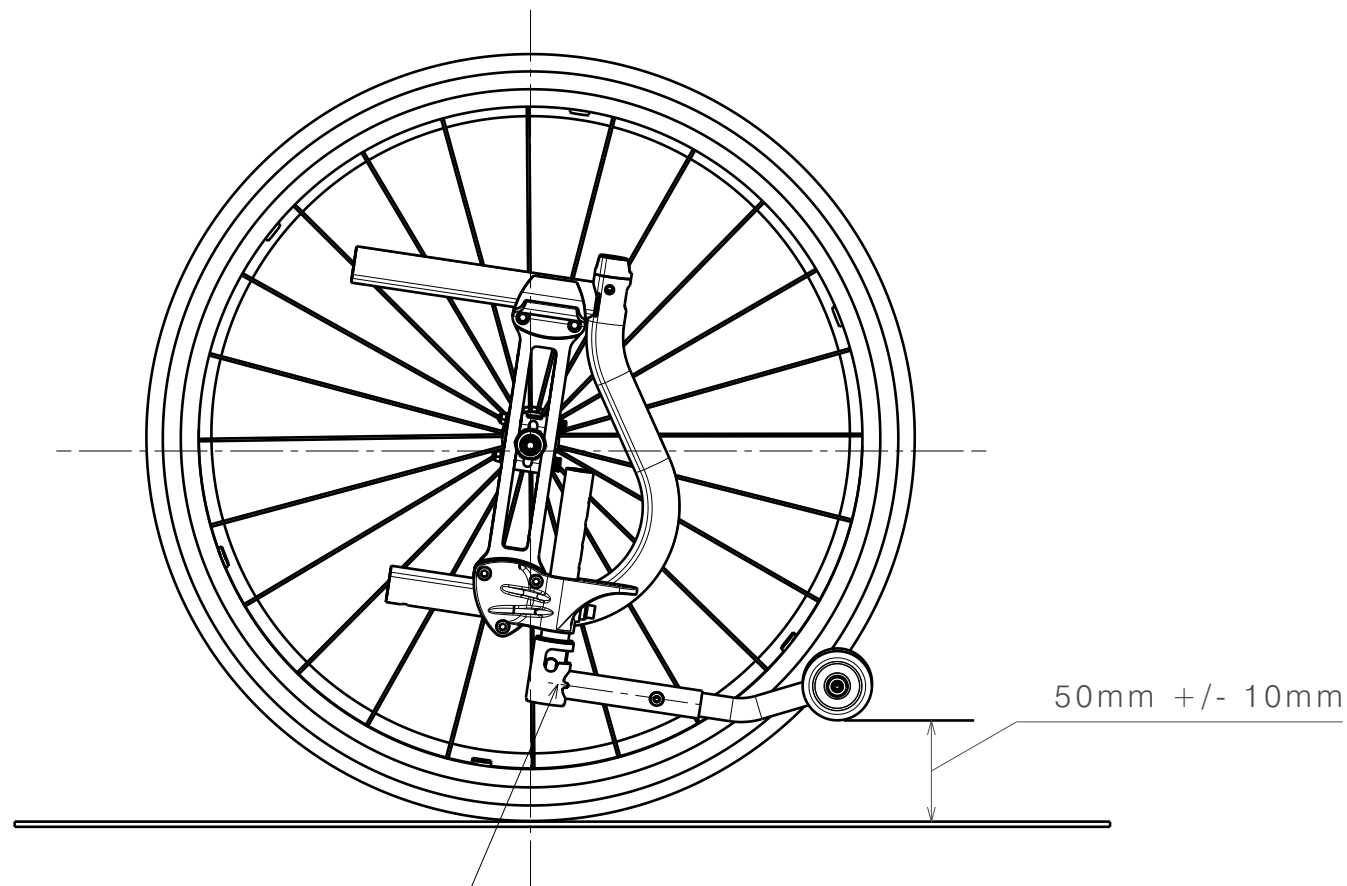

When the rear wheel is mounted in the highest position and the frame is very low, the anti-tipping wheel turns to the up.

/
Gdy tylne koło jest zamontowane w najwyższej pozycji, a rama jest bardzo niska, koło przeciwwyrotne obrotic do gory.

EP00297 / K

ANTI TIPPING FOR TRIGO				Page 3/3	Plan No.
					EP00297 / K
Scale:	1:4	Drafted	Verified	Confirmed	
Date	26/05/2018	06/02/2024	06/02/2024		
Initials	AS	PM	AS		