

Use WEICONLOCK AN 302-43.

Use WEICONLOCK AN 302-43.

Use WEICONLOCK AN 302-43.

Degrease the screws before using the adhesive.

Detail B
Scale: 3:5

Drawing shows only right side

EP00374 / I

BACKREST ADJUSTABLE ANGLE AND ADJUSTABLE HIGHT WITH FOLDING HANDLE TRIGO				Plan No. EP00374 / I
Scale: 3:10	Drafted	Verified	Confirmed	
Date	13/02/2019	07/04/2023	07/04/2023	
Initials	AS	PM	AS	

Material (35-40)			
No.	Plan	Article	Quantity
1	YP07083	2745665	1
2	YP08603	2622665	1
**			

Material (35-40) Without a handle.			
No.	Plan	Article	Quantity
1	YP07083	2745665	1
2	YP08604	2622765	1

Material (40-45)			
No.	Plan	Article	Quantity
1	YP07084	2745765	1
2	YP08603	2622665	1
**			

Material (40-45) Without a handle.			
No.	Plan	Article	Quantity
1	YP07084	2745765	1
2	YP08604	2622765	1


Material (45-50)			
No.	Plan	Article	Quantity
1	YP07084	2745765	1
2	YP08604	2622765	1
**			

Material (45-50) Without a handle.			
No.	Plan	Article	Quantity
1	YP07084	2745765	1
2	YP08605	2622865	1

Material			
No.	Plan	Article	Quantity
3	XP03203	GP25700	1
4	XP01862	GP13200	1
5	M4x12	1907582	2
6	XP01632	GP09399	1
7	XP01861	GP12900	1
8	M5x12 DIN 7991	2994076	1
9	M6x30	1992721	2
10	M6	2994059	1
11	WASHERM6	2994068	1
12	Fold. push handle	EP00246	1
13	Y004353	F000411	1
14	M6x10 DIN916	2994284	2
15	M6X10	2994127	2
*16	Cap D22	1990648	1
17	M6X6 DIN916/45H	1998138	2

* When unused folding handle.

** When the folding handle is not used, the backrest is 4cm lower.

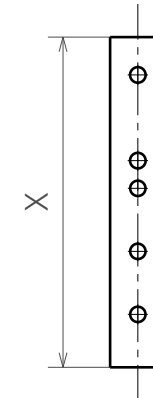

Plan	X
YP08603	144mm
YP08604	193,5mm
YP08605	243,5mm

EP00374 / I

BACKREST ADJUSTABLE ANGLE AND ADJUSTABLE HIGHT WITH FOLDING HANDLE TRIGO Page 2/2				Plan No.
Scale: 3:10	Drafted	Verified	Confirmed	EP00374 / I
Date	13/02/2019	07/04/2023	07/04/2023	
Initials	AS	PM	AS	