
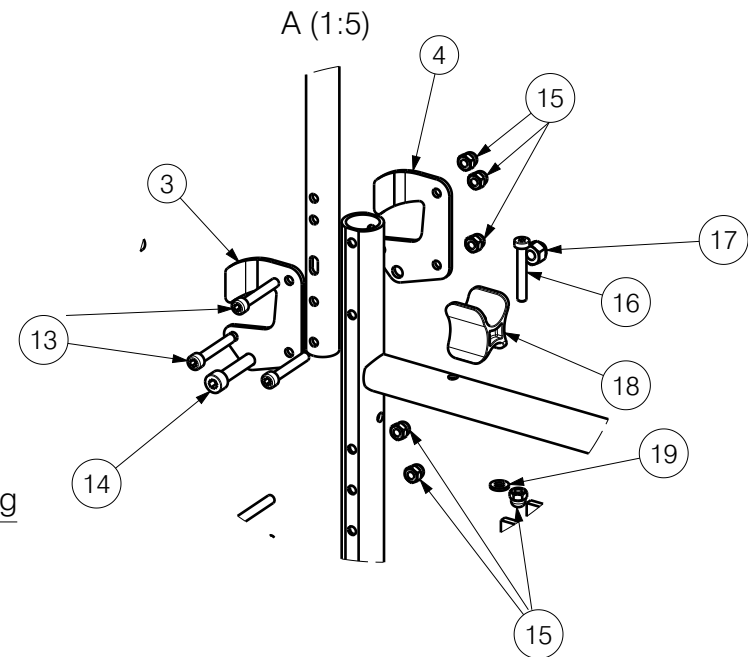


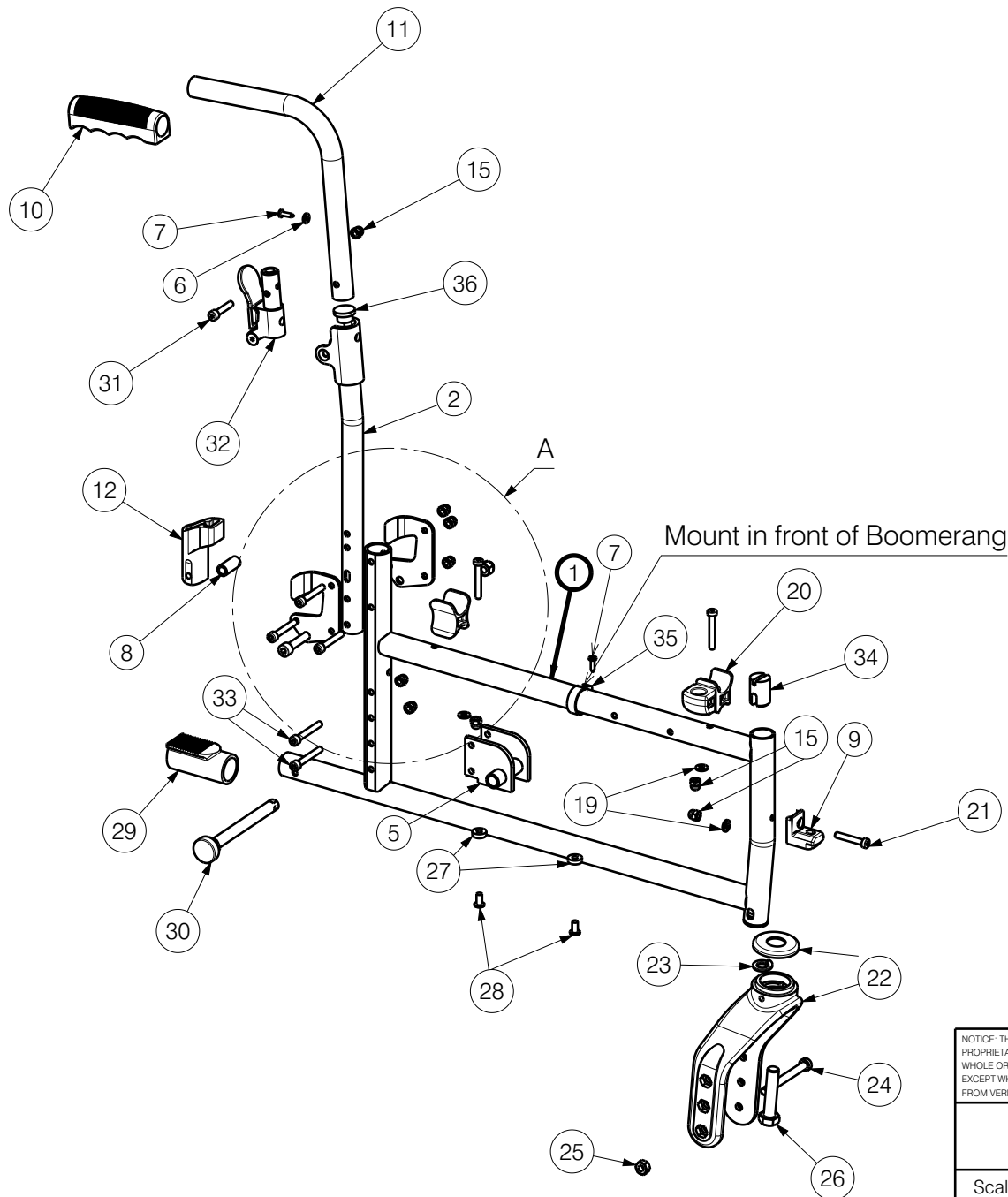

*Item 26 add torque 60 Nm

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

1/2

--

SIDE FRAME FOR D200 B69

Plan No.

EC00193/J

Scale: 1:7	Drafted	Verified	Confirmed
Date	14/08/2013	31-01-2025	31-01-2025
Initials	BZ	MP	TK



No.	Plan	Article	Qty
1	YC01481	1C475	2
2	YC01361	1C788	2
3	Y016063	1736154	2
4	Y016062	1736054	2
5	YC00657 (L)	1C24754	1
	YC00658(R)	1C24854	1
6	--	1001550	2
7	--	1003021	4
8	X003204	1585100	2
9	X004415	17193	2
10	--	3990041	2
11	YC00030	1C009	2
12	X004487	1904581	2
13	--	1003909	6
14	--	1001500	2
15	--	1008492	18
16	--	1006850	4
17	--	1902024	2
18	X004309	1904133	2
19	--	1001564	8
20	X004420	1904134	2
21	--	1903903	2
22	--	3990495	2
23	--	1001572	2

No.	Plan	Article	Qty
24	--	1998993	2
25	--	1998994	2
26	--	3994158	2
27	X004973	1907519	4
28	--	1992835	4
29	X004469	1905045	2
30	XC00037	3990106	2
31	--	1001495	2
32		3991658	2
33	--	1003109	4
34	X004818	1906184	2
35	XC00234	3993197	2
36	--	3990684	2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

2/2

--

SIDE FRAME FOR D200 B69

Plan No.

EC00193/J

Scale:	--	Drafted	Verified	Confirmed
Date		14/08/2013	31-01-2025	31-01-2025
Initials		BZ	MP	TK

