


Front side

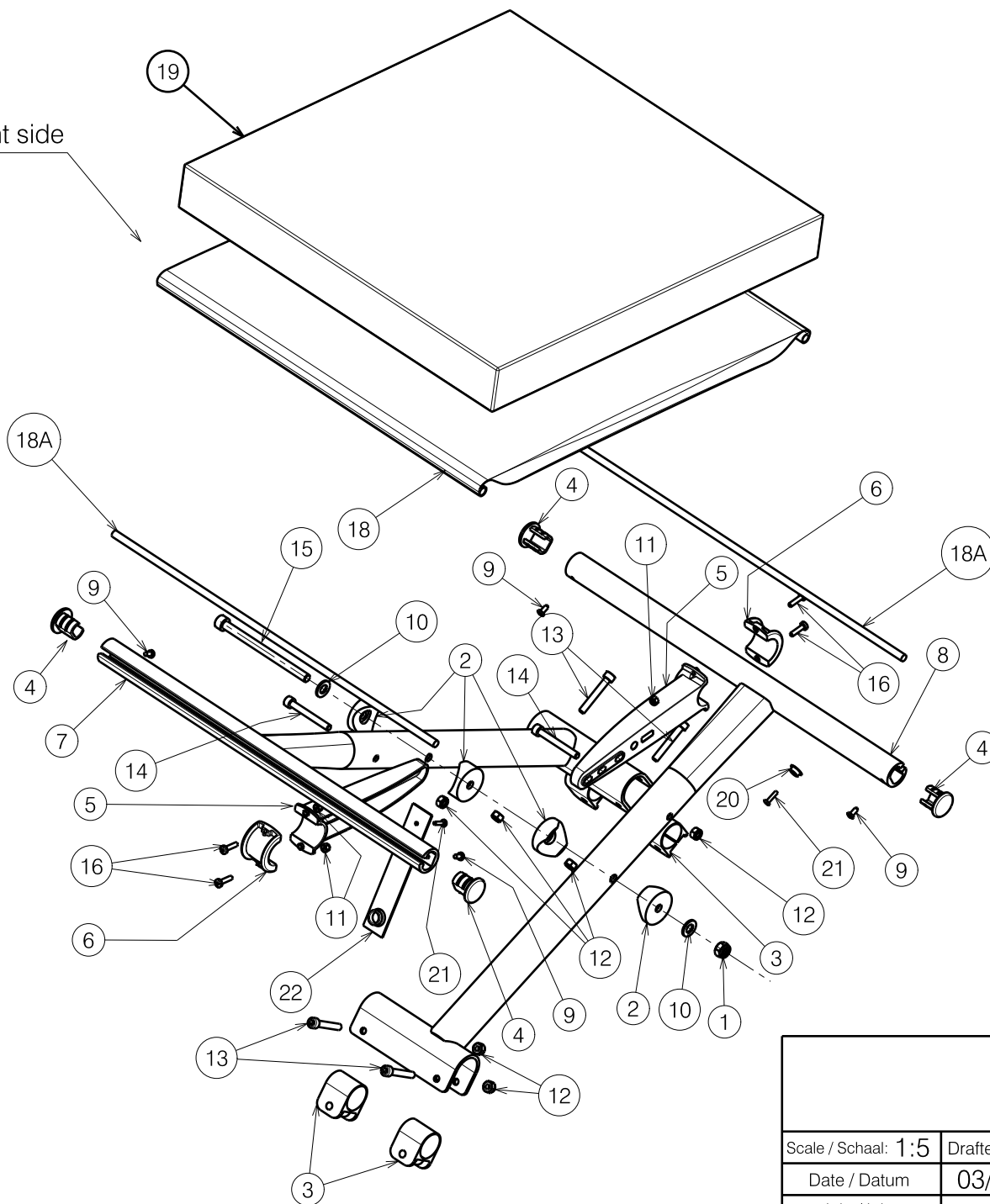

Sheet 1/2

2991720

CROSS V300 DL KRUIS V300 DL				Plan No. / Plannr. E001012/ E
Scale / Schaal: 1:5	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	03/12/2015	13/01/2025	13/01/2025	
Init. / Init.	SD	SD	PJ	

Material							
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	X001016	1001682	1	18	---	1911972 (39)	1
2	X002133	1992424	4		---	1911973 (42)	1
3	X003692	1998763	4		---	1911974 (44)	1
4	X004452	1904058	4		---	1911975 (46)	1
5	X004606	1905091 (39/44/48)	2		---	1911976 (48)	1
	X004607	1905092 (42/46/50)	2		---	1911977 (50)	1
6	X004620	1905090	2	19	B24 Seat Depth 42	1911963 (39)	1
7	Y015110	1728754 (39)	1			1911964 (42)	1
	Y015113	1728954 (42)	1			1911965 (44)	1
	Y015099	1729154 (44)	1			1911966 (46)	1
	Y015116	1729354 (46)	1			1911967 (48)	1
	Y015119	1729554 (48)	1			1911968 (50)	1
	Y015122	1729754 (50)	1		B24 Seat Depth 48	1922135 (39)	1
8	Y015111	1728654 (39)	1			1922136 (42)	1
	Y015114	1728854 (42)	1			1922137 (44)	1
	Y015100	1729054 (44)	1			1922138 (46)	1
	Y015117	1729254 (46)	1			1922139 (48)	1
	Y015120	1729454 (48)	1			1922140 (50)	1
	Y015123	1729654 (50)	1	20	---	1900773	1
9	---	1001530	4	21	---	1905708	2
10	X001119	1001554	2	22	---	1905711	1
11	---	1001678	4				
12	---	1001680	6				
13	M6x40 DIN912	1003909	4				
14	M6x50 DIN912	1007970	2				
15	---	1991711	1				
16	---	2990185	4				
18A	---	H001126	0.84				

Sheet 2/2

2991720

CROSS V300 DL				Plan No. / Plannr.
KRUIS V300 DL				E001012/ E
Scale / Schaal: 1:5	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	03/12/2015	13/01/2025	13/01/2025	
Init. / Init.	SD	SD	PJ	